

# Installation and Adjustment Instructions

## Powered Cart with Advanced Interface

### (Automatic Height Adjustment)



## Quick Setup

## Medical Facility Responsibility

**WARNING:** This Quick Setup guide is not a replacement for the full manual. It is the responsibility of the end user to ensure all aspects of the installation are covered by following the full manual.

**NOTE:** Inspect cart integrity prior to beginning setup.

### 1. First Time Initializing:

Briefly plug cart into power outlet – Advanced Interface should turn on (may take up to 30 seconds).

After initializing the cart, a “no Home” symbol (🏠) will appear in the top right hand corner of the Advanced Interface.

This means the user must set the cart’s “Home” (lowest) position.

### 2. Setting Home Position:

1. Hold the Down arrow to lower the cart’s worksurface until the collision symbol (⚠️) appears.
2. Release the Down arrow and reverse direction to raise worksurface to desired position.

### 3. Setting Time and Date:

1. Tap on “GUEST” in top left corner of interface (**Figure 1**).
2. Select “ADMIN” from drop down list and login using password: AMICO (**Figure 2**).
3. Tap on “Settings” icon.
4. Select “Admin Setup” (only available in the ADMIN profile).
5. Select “Set Clock”.
6. Using the arrow icons set the time and date in the desired format (note: date and time formats must first be selected on right of interface). Tap save to finish.

### 4. Creating a User Profile:

1. Tap on “ADMIN” in top left corner of interface and select “Logout” when prompted.
2. Tap “GUEST” in top left corner of interface, and select “New User”.
3. Enter desired User Name, select Next.
4. Enter desired Password, select Next (re-enter password and select Next).
5. Enter the user’s height. Select Next then OK when prompted.

### 5. Setting “Sit” Heights:

Note: User’s forearms should be parallel to ground (90-degree bend at elbows) for ergonomic use.

1. To input sit and stand positions, select Settings and then User Setup (**Figure 3**).
2. Adjust the height using the Up/Down arrows. When height is at desired sit position, select Save next to respective position icons.

### 6. Setting “Stand” Heights:

1. To input sit and stand positions, select Settings and then User Setup (**Figure 3**).
2. Adjust the height using the Up/Down arrows. When height is at desired stand position, select Save next to respective position icons.
3. Tap OK twice to return to Main Menu.

For power system software download, please visit <http://www.amico.com/hummingbird-power-system>

Amico Accessories Inc. 122 East Beaver Creek Road, Richmond Hill, ON L4B 1G6, Canada  
Toll Free Tel: 1.877.264.2697 | Tel: 905.763.7778 | Fax: 905.763.8587  
[www.amico.com](http://www.amico.com)

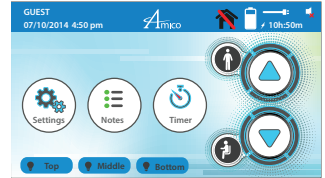


Figure 1



Figure 2



Figure 3

# Installation and Adjustment Instructions

## Powered Cart with Advanced Interface

### (Manual Height Adjustment)



## Quick Setup

## Medical Facility Responsibility

**WARNING:** This Quick Setup guide is not a replacement for the full manual. It is the responsibility of the end user to ensure all aspects of the installation are covered by following the full manual.

**NOTE:** Inspect cart integrity prior to beginning setup.

### 1. First Time Initializing:

Briefly plug cart into power outlet – Advanced Interface should turn on (may take up to 30 seconds).

### 2. Setting Time and Date:

1. Tap on “GUEST” in top left corner of interface.
2. Select “ADMIN” from drop down list and login using password: AMICO (**Figure 1**).
3. Tap on “Settings” icon.
4. Select “Admin Setup” (Only available in the ADMIN profile).
5. Select “Set Clock”.
6. Using the arrow icons set the time and date in the desired format (note: date and time formats must first be selected on right of interface). Tap save to finish.

### 3. Creating a User Profile:

1. Tap on “ADMIN” in top left corner of interface and select “Logout” when prompted.
2. Tap “GUEST” in top left corner of interface, and select “New User”.
3. Enter desired User Name, select Next (**Figure 2**).
4. Enter desired Password, select Next (re-enter password and select Next).

### 4. Height Adjustment:

1. Pull lever and adjust to desired height (**Figure 3**).



Figure 1



Figure 2

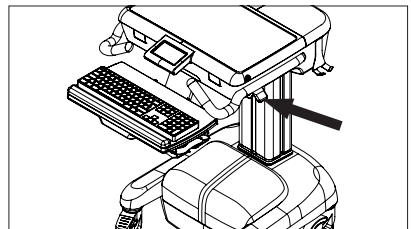


Figure 3

For power system software download, please visit <http://www.amico.com/hummingbird-power-system>

Amico Accessories Inc. 122 East Beaver Creek Road, Richmond Hill, ON L4B 1G6, Canada  
Toll Free Tel: 1.877.264.2697 | Tel: 905.763.7778 | Fax: 905.763.8587  
[www.amico.com](http://www.amico.com)